REFLECTION

PORTUGUÊS

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ENGLISH

1. Product identification

Reflection is a condensation-type silicone-based impression material for use in dentistry or dental prosthetics. It comes in the following versions: Putty (High Consistency), Light Body (Low Consistency) and Catalyst (Catalyst), allowing the condensation polymerization reaction to form a polymer rich in cross-links and highly stable dimensionally and chemically. Reflection has excellent elastic recovery and compatibility with dental plasters. It is used to obtain precision molds due to its outstanding characteristic of excellent reproduction of anatomical details.

2 Composition:

-Reflection Putty: polydimethyl siloxane; corn starch; mineral oil; petroleum jelly; talc; zinc oxide; silicon dioxide; calcium silicate; flavorings; dye; water.
-Reflection Light Body: polydimethyl siloxane; calcium silicate; silicon dioxide; calcium silicate; colorant; surfactant.
-Reflection Catalyst: petroleum jelly; mineral oil; metatin; silicon dioxide; silicone hardener; colorant.

3 Indications:

-Impression of dental arches or for edentulous patients, with a view to making models for total, partial or single dental prostheses.
-Double impression or simultaneous impression techniques
-Copper ring technique
-Impressions for study models, orthodontic models, matrices, protective isolation of the teeth during the placement of the prosthesis in the mufla.

4. Instructions for use:

4.1. Double impression

- Select a rigid tray with the capacity to provide a thickness of 3 to 4 mm of impression material. If necessary, test on the patient to be molded.
- Clean and dry the field and, if necessary, remove the gums using a gingival retractor wire.
- Using the measuring spoon, remove the desired amount of Reflection Putty from the pot, making sure that the measuring spoon is properly filled, with no empty spaces or excess material.
- Press the Reflection Putty against the palm of your hand and mark the diameter of the measuring spoon against the Reflection Putty.
- For each spoonful of dense putty used, distribute Reflection Catalyst in a proportion equivalent to the diameter marked.
- Mix for 30 seconds with your fingertips until you obtain a uniform color and a homogeneous mass.
- Place the mixture in the tray and position the tray in the desired location, applying pressure for 4 to 5 seconds.
- NOTE: Before molding with Reflection Putty, relieve it with PVC film or any other plastic, or after molding, relieve it with a sharp instrument, or move the tray around to relieve it after 1 (one) minute of insertion with the material.
- The waiting time will be approximately 3 minutes.
- Remove the tray from the oral cavity.
- Dispense the required amount of Reflection Light Body onto a glass plate, taking care that the material is dispensed in a straight line.
- For each brand of Reflection Light Body, dispense the same amount of Reflection Catalyst. Using more catalyst speeds up setting.
- Incorporate the fluid mass into the catalyst, spatulating vigorously so as to homogenize all the material within 30 seconds.
- Using a molding syringe, pour part of the material into the cavity preparation. The rest of the material should be dispensed into the first impression with the Reflection Putty, after relieving it.
- Position the tray at the desired location, applying pressure for 4 to 5 seconds and keep it stable for another 4 minutes before removing it from the oral cavity.
- After removing the tray, rinse the mold with running water and disinfect before pouring the model.

4.2 Simultaneous impression taking

- Select a rigid tray with the capacity to provide a thickness of 3 to 4 mm of molding material. If necessary, test it on the patient to be molded.
- Clean and dry the field and, if necessary, remove the gums using a gingival retractor wire.
- Using the measuring spoon, remove the desired amount of Reflection Putty from the pot, making sure that the measuring spoon is properly filled, with no empty spaces or excess material.
- Press the Reflection Putty against the palm of your hand and mark the diameter of the measuring spoon against the Reflection Putty.
- For each spoonful of dense putty used, distribute Reflection Catalyst in a proportion equivalent to the diameter marked.
- Mix the Reflection Putty with the Reflection Catalyst for 30 seconds with your fingertips until a uniform color and homogeneous mass are obtained.
- At the same time, the assistant should dispense the required amount of Reflection Light Body onto the glass plate, taking care that the material is dispensed in a straight line.
- For each brand of Reflection Light Body, dispense the same amount of Reflection Catalyst. Using more catalyst speeds up setting.
- Start mixing the fluid mass with the catalyst, spatulating vigorously so as to homogenize all the material in no more than 30 seconds.
- Place the mixture inside the tray with the Reflection Putty material after manipulation.
- Using a molding syringe, inject the mixture around the cavity preparation.
- Position the tray at the desired location, applying pressure for 4 to 5 seconds and keep it stable for another 4 minutes before removing it from the oral cavity.
- After removing the tray, rinse the mold with running water and disinfect before pouring the model.

TYPE	REFLECTION PUTTY
MIXING TIME	30’’
WORKING TIME	1’30’’
WAITING TIME AT THE MOUTH	3’’
SETTING TIME	4’30’’

TYPE	REFLECTION LIGHT BODY
MIXING TIME	30’’
WORKING TIME	1’30’’
WAITING TIME AT THE MOUTH	4’’
SETTING TIME	5’30’’

4.3 Disinfection:

The impression after being washed in water and can be disinfected with any suitable chemical dental disinfectant solution on the market.

5. Precautions, Warnings and Contraindications:

- Precautions and Warnings
-FOR PROFESSIONAL USE
-All dental materials must be used exclusively by dentists or authorized laboratories.
-The setting time given is for a temperature of 23°C. Increasing the temperature speeds up the working time, while decreasing the temperature lengthens the setting time.
-Increasing the amount of Reflection Catalyst reduces the working time.
-Keep the product out of the reach of children.
-When disposing of the material, proceed with mixing the base and catalyst so that the material polymerizes before disposal.
-Disposal of the material must be carried out in accordance with laboratory and/or dental office waste disposal and/or the regulations in force at each location.
-Reflection Catalyst is irritating. Avoid direct contact with the oral mucosa.
-The use of PPE (goggles and gloves) is recommended when using the product.
-In case of contact with eyes, rinse with plenty of running water and seek medical attention.
-In-use product
Contraindications
There are no contraindications when the product is used by dentists or authorized laboratories, in accordance with the instructions for use. However, patients with a history of allergy to one or more components of the product are advised not to use it.

6. Shelf life:

- The expiry date represents the maximum period for use of the product.
- Do not use after the expiration date.
- Under proper storage conditions, Reflection has a shelf life of 36 months, even after opening.
- 7. Storage:
 - Store at room temperature, between 15°C and 30°C, in a dry place protected from light and heat.
 - Keep packaging closed after use.
 - Keep the product in its original packaging
 - Inadequate storage conditions reduce shelf life and lead to product malfunction
 - If stored under refrigeration, allow the product to reach room temperature before use.

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