IFU-OTH-0014

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Revisão / Revision: 0
Data / Date: 27/02/2024

**INSTRUCTIONS FOR
USE**

Technical Name: ORTHOMETRIC ELASTOMERICS

Product Name: ORTHOMETRIC ELASTOMERICS

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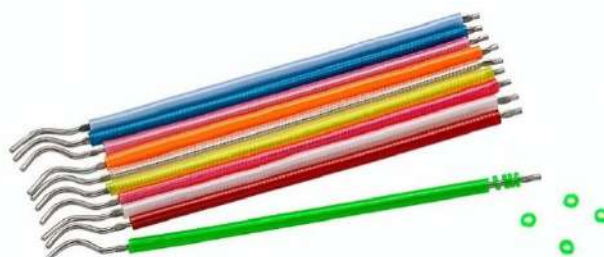
- Telemóvel: +351 967 417 654

Product description

The modular elastics and stick ligature are made of TPU polymer (Thermoplastic Polyurethane) and have a design that establishes a good adaptation to the tooth movement elements, providing the patient with comfort, quality, safety and reliability.

When used together with brackets, tubes and accessories fixed on the surface of the tooth, the elastics aim to help in the fixation or generation of forces, contributing to establish the mechanical-biological system, and in this way make it possible to apply force to the teeth, which results in induced tooth movement, correcting malocclusion.

Intended use

The Orthometric elastic bandage is made up of a polymer-based elastic ring, which is available in various colors and in two different forms: elastic rings arranged on a metal support structure, called a cane; or modular, in which the elastic rings are arranged separately on a main polymer structure. The basic difference between one situation and the other is that modular elastics are more hygienic and can be used on a single patient, since 22 elastics make up a modular structure.

The elastic bandages have the property of returning to their original shape, with a certain deformation of the material, and are stretched around the bracket fins, promoting the fixation of the orthodontic arch to the

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accessory channel, preventing its displacement, and thus enabling the application of the force necessary for tooth movement.

How to Use

- For the proper use of stick ligature, it is recommended to:

- 1) Separate the elastic rings;
- 2) Applying the elastic with the Sputnik instrument, which has two active tips, one tapered and the other curved, forming a handle;
- 3) The elastic is stretched and adjusted at the curved end;
- 4) The elastic is guided to the bracket and a rotating movement from cervical to occlusal is promoted;
- 5) Check that the elastic has adapted perfectly to the bracket, involving all the flaps;
- 6) Occasionally, the elastic can be guided using other instruments, such as a needle holder or a gun-shaped guiding instrument.
- 7) Advise the patient on hygiene care, as the elastic may encourage the accumulation of food residues and cause some irritation to soft tissue, or even cause stains if the problem persists routinely.

It is worth noting that the elastic can break or become dislodged from the bracket straps, and the patient should be advised to notify the professional if this type of problem occurs.

Instructions for Removing the Elastics

To remove the elastic, use the tapered active tip of the Sputnik instrument and disengage the elastic from the bracket fins. The elastic must be disposed of properly as it contains food residues and micro-organisms, avoiding contamination of the professional and assistant.

Some warnings can be listed when using elastics:

- 1) Avoid using elastic whose packaging has been tampered with, to avoid the risk of cross-contamination between professionals and patients;
- 2) Make sure the elastic is original and within its expiration date;
- 3) Elastics should be stored in a suitable environment, away from heat, dry environments and sunlight;
- 4) Advise the patient on proper hygiene;
- 5) Allergic reactions can occur when using elastics and the patient should be instructed to report this;
- 6) The professional should warn the patient of the risks of using elastic: tooth sensitivity; possible allergic reaction and tooth decalcification when the area is not properly cleaned; and
- 7) Any undesirable effect resulting from the use of elastic is considered a professional responsibility.

Composition:

Material	Concentration
Thermopolyurethane	<97%
Masterbatch	>3%

Storage Conditions:

In intact and closed packaging.

This product must be stored in a clean, dry environment, away from heat sources and sunlight, as inadequate conservation of the elastics may compromise the properties of the material, impairing the functioning of the product.

The manufacturer recommends the single use of this product. It should be disposed of correctly in accordance with the information contained in the Manual of Dental Services Risk Prevention and Control, Technology in Health Services, 2006 (available on the ANVISA website).

Warnings and Symbology:

- It is essential that only the specialist installs the product on the patient, since in addition to the clinical examination, the professional must have the necessary knowledge to make a systematic description of the patient's dentofacial features, list the priority problems, describe possible solutions for each of the problems, considering the possible favorable or unfavorable interactions for each of the proposed attempts.
- It is also the orthodontist's duty to determine the maximum expected gain from orthodontic therapy and the minimum loss, or to calculate the benefit versus risk ratio for each patient, with both risk and benefit including psychosomatic factors to determine the best management for each patient.
- Patients should be instructed in oral hygiene, maintenance, and food care. The following should be avoided: ice, coconut, hard and/or sticky sweets, caramels, nougats, jellybeans, corn, popcorn, and sugar cane. Biting hard objects such as pens and pencils has the same effect. Chewing gum and toffee often loosen the bands or rings that have been cemented in place.
- The product is supplied in non-sterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are required, provided the packaging has not been tampered. If the product comes into contact with surfaces or substances other than those intended for use, it is recommended that it be discarded.
- The product must not be reused or reprocessed. Do not use the product if its packaging is damaged or open, to minimize possible dangers related to the transmission of microorganisms or possible product counterfeiting.
- It is the responsibility of qualified professionals to advise their patients of the risk of aspiration and swallowing in the event of breakage/ detachment of parts, do not to underestimate the possible immediate or delayed complications resulting from this type of accident and to adequately inform the patient and/or their legal guardians of the risks inherent in the event and the measures to be taken.
- This product should only be used by qualified orthodontic professionals. The use by people without

the necessary technical knowledge could lead to unwanted tooth movement, bone loss, tooth loss, fenestration, gingival recession, or root dehiscence.

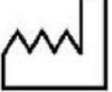
- It is the responsibility of the qualified professional to advise the patient on correct oral hygiene, in order to avoid the appearance of plaque and tartar or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis. Considering that the product is used in contact with mucous membranes and body fluids, it is recommended that professionals apply good clinical practices for the disposal of medical products in accordance with current health regulations.

ATTENTION: Orthometric cannot be held responsible for the results obtained by applying the product without the necessary precautions or failing to observe the warnings.

Contraindications

Orthometric only supplies its products to qualified professionals. It is the orthodontist's sole responsibility to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene.
- Patients unable to cooperate with treatment.
- Patients with a disease or limitation that makes the treatment difficult.

	Do not re-use		Authorized representative in the European Community
	Consult instructions for use		European conformity
	Medical device		Caution
	Catalogue number		Non-sterile
	Batch code		Contains Nickel
	Manufacturer		Date of manufacture

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Revisão / Revision: **0**
Data / Date: 27/02/2024

	Use-by date		Keep away from sunlight
	Keep dry		Do not use if package is damaged