DESCRIPTION

longlass Photo is a light-cured glass ionomer composite. It is an ideal material for dental applicatios in cavity bases, pit seals, sealing of extended pits and restoration of small lesions. Its chemical composition allows high adhesive strength in enamel and dentin layers, besides presenting excellent mechanical strength. longlass Photo is available in a single-component syringe, making it ideal and practical for handling in dental clinics, in addition, its pigmentation with universal opaque and blue color systems allows clinical applications for different dentistry and endodontics works.

INDICATION OF USE

longlass Photo is indicated for cavity bases in all restorative materials, sealing of extended pits and fissures and restoration of small lesions.

PRESENTATIONS

Available in colors: Opaque Universal and Blue.
1 Syringe with 2.5 g of light-curing fluid material and 5 application tips.

COMPOSITION

Methacrylic monomers, initiators, co-initiators, preservatives, pigments, fillers and glass ionomer filler.

MODE OF USE

1. Prepare the cavity according to the restauration technique that will be applied.
2. Wash and dry the tooth surface.
3. The operative field must be kept free of contamination. Use absolute isolation or effective relative isolation.
4. In very deep cavities, it is recommended to use a calcium hydroxide-based liner.
5. Apply longlass Photo directly into the cavity using the application tip attached to the syringe and light cure the product for 20 seconds (It is recommended to use photoactivation devices with at least 1000 mW/cm²).
6. For cavity bases thicker than 1 mm, apply and cure in layers.
7. The light guide should be placed as close as possible to the surface (approx. 2 mm) where the material is to be applied. For a larger distance, extend the polymerization time.
8. After use, discard the application tip to avoid contamination.
9. At the end of the use of the product, slightly retract the syringe barrel to avoid overflow of the product.

longlass Photo Blue

To use this product as a marker for the area to be lined with glass ionomer, follow the same procedures described above. However, the product should only be used as a guide after endodontic treatments in deep cavity bases, as it has no aesthetic appeal.

WARNINGS

- Exclusive use of dental surgeon.
- Do not reuse empty packaging.
- Keep out of reach of children.

- This material was manufactured for dental use only and must be handled according to the instructions for use. The manufacturer is not responsible for damage caused by other uses or by incorrect handling not described in this manual. Furthermore, the user is obliged to verify, before use and under his/her own responsibility, if this material is compatible with the desired use, especially when this use is not indicated in this instructions for use.
- Strictly follow the procedures provided in the instructions for use.
- Do not allow objects, pastes and loose polymerized parts to be swallowed or aspirated.
- Contact between unpolymerized longlass Photo and skin, mucous membranes and eyes should be avoided. When not yet polymerized, the product can have a slightly irritating effect.
- In case of allergic reactions, discontinue use immediately. Seek medical advice if necessary.
- Single use product. After photoactivation, the product does not return to its original consistency.
- Reprocessing prohibited.
- Do not expose the material to temperatures above 25 °C.
- Do not expose the material directly to sunlight.
- Replace the syringe cap immediately after use.
- Avoid contact with the eyes to avoid damaging the cornea. In the event of a reaction with this tissue, rinse with plenty of water and seek medical attention.
- Avoid contact with the oral mucosa in order to prevent the formation of inflammatory processes. In the event of accidental contact, rinse with plenty of water and seek medical attention.
- Avoid ingesting the product. In accidental cases, the patient should drink plenty of water and, in the event of unwanted reactions, medical assistance should be sought.
- Do not use the material if the packaging is damaged.
- longlass Photo Blue should not be used in aesthetic finishes, as its color is not suitable for natural teeth.

PRECAUTIONS

- This product should only be used in accordance with the instructions for use.
- Wear PPE when handling the product, including gloves and goggles. It is recommended that patients also wear goggles.
- Contaminated syringes should be discarded.
- Syringes cannot be reprocessed and must be handled with disinfected gloves.
- A new application tip should be used for each patient to avoid contamination. Discard it after each use.
- In the case of storage in refrigerated systems, the material should be left at room temperature before application so that it reaches the appropriate fluidity.

CONTRAINDICATION

longlass Photo should not be used in case of known hypersensitivity (allergy) to any of the components of the formulation.

ADVERSE EFFECT

Until now, no adverse effects of the longlass Photo product have been identified. However, there are reports in the literature of allergic reactions involving dental products based on methacrylic monomers.

STORAGE CONDITIONS

Store longlass Photo in its original packaging in a clean, dry place. Keep the material properly sealed and away from light or high humidity. The product should be kept at a controlled temperature between 4 °C and 25 °C.

TRANSPORT CONDITIONS

The product can be transported between -28 °C and 55 °C for up to 15 days without compromising its initial performance.

VALIDITY

If stored according to the manufacturer's instructions, the product has a shelf life of 24 months from the date of manufacture. Do not use after expiration date.

DISPOSAL

Do not reuse the product and dispose of in accordance with local legislation.

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