Sugador Cirúrgico

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Surgical Aspirator Tips

USE INSTRUCTIONS

SURGICAL ASPIRATOR TIPS: Two different diameters tips that allow suction in hard-to-access areas

INDICATIONS: Blood and fluid suction during surgeries.

USE INSTRUCTIONS: Connect the Surgical Aspirator Tips to the dental equipment suction hose and activate it. After using it discard the aspirator and unused tips. Do not reuse the product.

STORAGE: Keep the product in an unique and closed cabinet, protected from dust and insects. Do not expose to sun and heat.

COMPOSITION: **Plastic tube:** Non-toxic PVC (phthalate free). | **Tips:** Polyethylene and non-toxic pigments.

WARNING: Do not use the product if the package is damaged, open or damp. Do not use the product after the expiration date of the sterilization indicated on the package. Single use, disposable product and reprocessing prohibited. Do not reuse.

SHELF LIFE: 02 years from manufacturing date in original non-violated package. Do not use the product after the indicated expiring date. **DISPOSAL:** Disposal of this product should be done in accordance with local environmental legislation. After opening the package and after performing the surgical procedure discard even the tips that are not used.

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