Description

OptraGate® is an auxiliary aid for clinical use which allows the working field to be accessed with ease.

It provides retraction of lips and cheeks during dental treatment, offers the clinician increased visibility and accessibility and facilitates moisture control in the oral cavity. The device, which features high three-dimensional flexibility and elasticity, ensures patient comfort and assists in keeping the patient's mouth open. OptraGate is available in sizes Regular, Small as well as Junior for children between 5 and 10 years of age.

Composition

Thermoplastic elastomer (SEBS) and polypropylene.

OptraGate is latex-free.

Indication

Suitable for use in diagnostic, preventive and therapeutic dental procedures (in particular restorative procedures, cavity preparation, professional tooth cleaning, diagnostic procedures, scaling & root planing, tooth whitening, impression making).

Contraindications

The use of OptraGate is contraindicated

- in oral surgical and implantological procedures;
- if the patient is known to be allergic to any of the components of OptraGate.

Description of the procedure

The elastic material of OptraGate is stretched by mean of two rings. The thicker ring of the device is positioned in the area of the gingivobuccal fold, while the thinner ring, which features two tabs, remains outside the mouth. The elastic component between the two rings thus embraces the lips of the patient and provides retraction due to the restoring force of the rings (see Figure 1).

Fig 1: OptraGate in place with the tabs pointing downward. It does not interfere with the patient's occlusion.

1. Select the appropriate sized OptraGate. The size (R for regular, S for small or J for Junior) is printed on the package as well as on the left tab of OptraGate. To ensure optimum function, we recommend inserting the largest size that fits in the patient's mouth.

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2. OptraGate may be inserted from both the 12 o'clock position with the patient lying down, or from the 8 o'clock position with the patient sitting upright (or, the 4 o'clock position if the clinician is left-handed).
3. To ease placement, with the tabs pointing downward, the operator should grasp the thicker intraoral ring by reaching through the thinner extraoral ring. The inner ring should be held between the thumb and middle finger and pressed together slightly (see Figure 2).

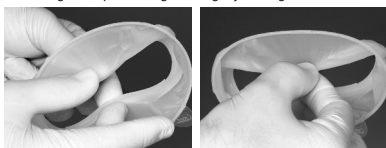
Fig 2: Correct hold on the intraoral ring with thumb and middle finger. The intraoral ring is slightly compressed in the process.

4. The slightly compressed intraoral ring is then inserted into the buccal corridor on one side, so that the elastic component embraces the corner of the mouth and the outer ring is still located extraorally (see Figure 3).

Fig 3: The thicker intraoral ring is positioned between the teeth and the corner of the mouth.

5. As soon as the OptraGate is securely in place on one side, the other side is inserted in the same manner by slightly bending it (see Figure 4).

Fig 4: The intraoral ring is positioned behind the two corners of the mouth.

6. Subsequently, the intraoral ring is placed behind the lower and upper lips and thus OptraGate assumes its final, stable position (see Figure 5). The intraoral ring can be inserted with more ease if the patient's mouth is relaxed.

Fig 5: Final positioning behind the lower and upper lip.

If, in individual cases, the intraoral ring should show a tendency to slip out of the gingivobuccal fold if the mouth is closed completely, positioning of the intraoral ring deeper inside the vestibule is usually sufficient to solve the problem. Sometimes, the use of another size may also be a solution.

The securely positioned OptraGate improves the overall view and accessibility of the working field. Lateral movements of the mandible and opening of the mouth to various degrees may provide additional space and access to the treatment area. OptraGate may even be left in place when checking occlusion.

7. To remove OptraGate, grasp the extraoral ring in the lower area with the help of a paper towel and lift it slightly so that the intraoral ring is loosened from the lower gingivobuccal fold (Figure 6). The intraoral ring can then be easily removed from the upper gingivobuccal fold and hygienically disposed of using the paper towel.

Fig 6: Removal of OptraGate

Notes

-  OptraGate is not sterile and thus not suitable for use with oral surgical or implantological procedures.
-  OptraGate is for single use only.
- It cannot be reused. The material is not suitable for sterilization. Cleaning and disinfecting agents may have a detrimental effect on the material and lead to undesirable reactions in the patient.
- When used in tooth whitening procedures, care should be taken that no bleaching material gets caught between OptraGate and the oral soft tissues to prevent an irritation of the skin and mucous membrane.
-  OptraGate should be stored at 2–28 °C / 36–82 °F.
-  These Instructions for Use are also available at the website of Ivoclar Vivadent AG (www.ivoclarvivadent.com).

Keep out of the reach of children!
For use in dentistry only!

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www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai

Trading Co., Ltd., 2/F Building 1, 881 Wuding Road, Jing An District

200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

www.ivoclarvivadent.de

Wieland Dental + Technik

GmbH & Co. KG, Schwenninger Strasse 13, D-75179 Pforzheim, Germany

Tel. +49 7231 3705 0, Fax +49 7231 3579 59

www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, A, ndheri (West), Mumbai, 400 053, India

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www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

www.ivoclarvivadent.pl

Ivoclar Vivadent Marketing Ltd.

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

P.O. Box 300146, Riyadh 11372, Saudi Arabia

Tel. +966 11 293 8345, Fax +966 11 293 8344

www.ivoclarvivadent.com

Ivoclar Vivadent Pte. Ltd.

171 Chin Swee Road, #02-01 San Centre, Singapore 169877

Tel. +65 6535 6775, Fax +65 6535 4991

www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.

Carretera de Fuencarral nº24, Portal 1 – Planta Baja
28108-Alcobendas (Madrid), Spain
Telf. +34 91 375 78 20, Fax +34 91 375 78 38
www.ivoclarvivadent.es

Ivoclar Vivadent AB

Dalvägen 14, S-169 56 Solna, Sweden
Tel. +46 8 514 939 30, Fax +46 8 514 939 40
www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office

: Tesvikiye Mahallesi, Sakayik Sokak, Nisantasi' Plaza No:38/2
Kat:5 Daire:24, 34021 Sisli – Istanbul, Turkey
Tel. +90 212 343 0802, Fax +90 212 343 0842
www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Ground Floor Compass Building, Feldspar Close, Warrens Business Park,
Enderby, Leicester LE19 4SE, United Kingdom
Tel. +44 116 284 7880, Fax +44 116 284 7881
www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA
Tel. +1 800 533 6825, Fax +1 716 691 2285
www.ivoclarvivadent.us