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Owner's Manual

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Jato de Bicarbonato - Jet Hand I

Bicarbonate Jet Jet Hand I

Chorro de bicarbonato- Jet Hand I

Cód. 300052009 Rev. 01

GNATUSO

ÍNDICE

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INDEX

English

PRESENTATION OF MANUAL	25
IDENTIFICATION OF EQUIPMENT	26
- Description of the Equipment	27
- Indication of Equipment	27
MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION	28
TECHNICAL SPECIFICATIONS	30
- Characteristics	30
- List of pieces and circuit scheme	30
- Standards applied	30
- Electromagnetic emissions.....	31
- Contents of the accessible and inaccessible markings	34
- Dimensions	35
- Symbologies of packaging	36
- Symbologies of product	36
OPERATION OF EQUIPMENT	37
PRECAUTIONS, RESTRICTIONS AND WARNINGS	38
- Conditions of transport and storage	38
- Sensitiveness to environmental conditions foreseeable in normal situations of use..	38
- Precautions and warnings "during the installation" of equipment.....	38
- Recommendations for preserving the equipment.....	39
- Precautions and warnings "during the utilization" of equipment	39
- Precautions and warnings "after" the use of equipment	39
- Precautions and warnings during the "cleaning and disinfection" of equipment.....	40
- Precautions in case of alteration in the functioning of equipment.....	40
- Recautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment.....	40
CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION.....	41
- Sterilization in Autoclave	41
- Disinfection	41
- Cleaning	41
- Additional procedures for reuse	42
- Preventive maintenance	43
- Corrective maintenance.....	43
UNFORESEEN EVENTS – SOLUTION OF PROBLEMS	43
WARRANTY OF EQUIPMENT	44
FINAL CONSIDERATIONS.....	44

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ATTENTION

For greater safety:

Read and understand all the instructions contained in these Instructions for Use before installing or operating this Equipment.

Note: These Instructions for Use must be read by all the operators of this Equipment.

IDENTIFICATION OF EQUIPMENT

Dear Customer

Congratulations. You have made a good choice when you decided to buy a GNATUS QUALITY product comparable to the best products available in the World. This manual is a general presentation of your product and it will give you important details to help you to solve possible problems.

Please, read it and keep this with you.

Identification

Technical Name: Bicarbonate Jet

Trade Name: Jet Hand I

Brand: GNATUS

IDENTIFICATION OF EQUIPMENT

Description of Equipment

Prophylactic equipment, Bicarbonate jet. Modern, compact design easy to use.

Composed of a body made of autoclavable anodized aluminum.

- Container of bicarbonate easy to access coupled to the handpiece;
- Autoclavable bicarbonate jet point.
- Irrigation with pneumatic system;

Internal pressurization via the high rotation terminal of the delivery unit, facilitating its functioning, dispensing with external air and water connections. Internal depressurization through automatic sweeping of the bicarbonate.

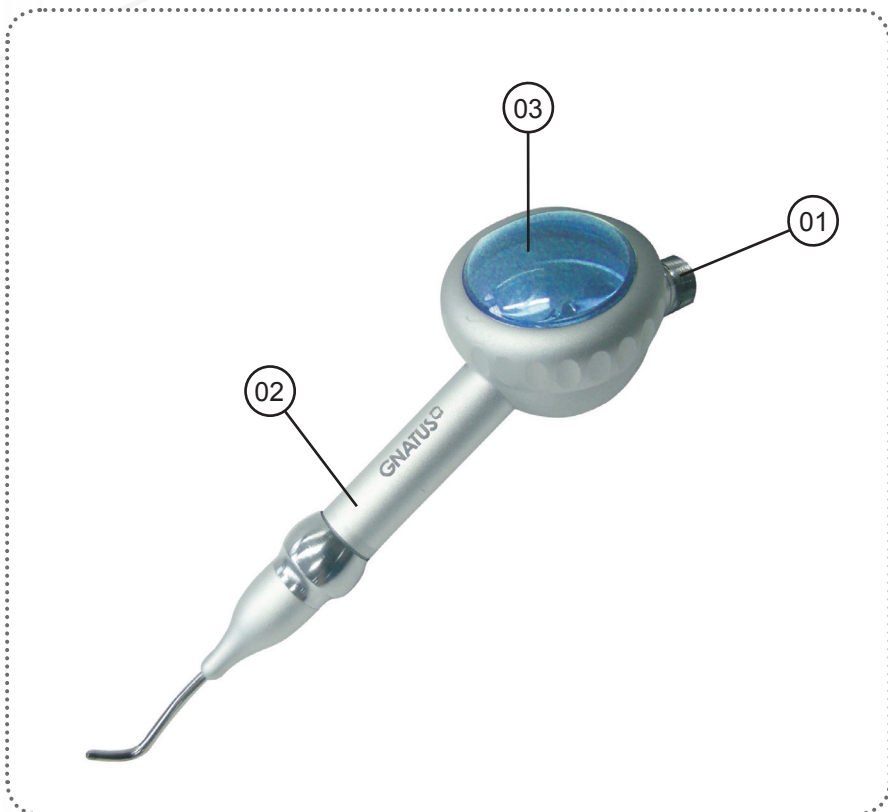
Water valve located in the triple terminal with spray, with sensitivity adjustment which allows adaptation to the requirements of each operation.

Indication of Equipment

This equipment is for dental use only. It must be operated and utilized by specialized professional (certified professional, according to the legislation of the country) and following the instructions of the manual.

The operation of the equipment required, for the professional, the utilization of correct instruments and it should to be in perfect conditions of the use, and to protect the professional, the patients and others, in the eventual danger situation.

MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION

01 - **Fast clamps - Bicarbonate Jet/Delivery Unit TB/TM**

02 - **Bicarbonate Jet hand piece**

03 - **Bicarbonate Reservoir**

MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION

Accessories which come with the Jet Hand

01 - **SACHET OF BICARBONATE** (EXEMPT OF REGISTRATION BY THE MINISTRY OF HEALTH)

02 - **REPAIR RINGS KIT**

03 - **COVERS FOR BICARBONATE RESERVOIR**

04 - **UNBLOCKING DEVICE**

The use of any part, accessory or material not specified or foreseen in these instructions for use is entirely the user's responsibility.

TECHNICAL SPECIFICATIONS

Characteristics

Product classification:
According standard NBR IEC60601-1
Operation mode
Continuous
Compressed air inlet pressure
30 to 40 PSI (adjusted at the terminal of the equipment)
Water inlet pressure
20 to 40 PSI (adjusted at the terminal of the equipment)
Operation
Through equipment pedal
Handpiece body
Anodized aluminum body
Net weight
0,145 kg
Gross weight
0,289 kg
Water leak protection
IPX 0

List of pieces and circuit scheme

Gnatus Company declares that the supply of the circuit scheme, list of pieces or any other information that propitiate technical attendance for the user, can be request if there is an agreement between the user and Gnatus Company.

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The materials used to produce the equipment are Biocompatible.

TECHNICAL SPECIFICATIONS

Electromagnetic emissions

Recommended distances between portable and mobile RF communication equipments and the Jet Hand I			
The Jet Hand I is made to be used in an electromagnetic environment in which RF disturbances are controlled. The client or the user of the Jet Hand I may help preventing electromagnetic interference by keeping a minimal distance between mobile and portable RF communication equipment (transmitters) and the Jet Hand I, as recommended below, in accordance with the maximal voltage output of the communication equipment.			
Transmitter Maximum Output (W)	Separation distance according to transmitter frequency (M)		
	150 kHz to 80 Mhz $d = 1,2\sqrt{p}$	80 kHz to 800° Mhz $d = 1,2\sqrt{p}$	800 kHz to 2,5° GHz $d = 2,3\sqrt{p}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
For transmitters with a maximum nominal output power not listed above, the recommended d separation distance in meters (M) can be determined using an equation applicable to the frequency of the transmitter, where P is the transmitter maximum nominal output in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, is applied the separation distance for the higher frequency range.			
NOTE 2 These guidelines may not apply to all situations. The absorption and reflection from structures, objects and people affect the electromagnetic propagation.			

TECHNICAL SPECIFICATIONS

Electromagnetic emissions

Guidelines and manufacturer's declaration - electromagnetic immunity			
The Jet Hand I is made to be used in the electromagnetic environments specified below. The client or the user of the Jet Hand I must be sure that it is used in such environment.			
Immunity test	ABNT test level NBR IEC 60601	Level of compliance	Electromagnetic Environment Directives
RF conducted IEC 61000-4-6 RF radiated IEC 61000-4-3	3 vrms 150 kHz up to 80 MHz 3 V/m 88 MHz up to 2,5 GHz	3 Vrms 3 V/m	It is advisable that portable and mobile RF communication equipment is not used near any part of the equipment, including cables, with a separation distance less than the one recommended, calculated from the equation applicable to the frequency of the transmitter: Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P} \text{ 80 MHz thru 800MHz}$ $d = 2,3 \sqrt{P} \text{ 800 MHz thru 2,5MHz}$ Where P is the nominal maximum power of output of the transmitter in watts (W), as per the manufacturer of the transmitter, and d is the recommended separation distance in meters (m). It is advisable that the field intensity from the RF, transmitter as determined by means of electric inspection on-site, ^a is less than the level of compliance in each frequency range ^b. There may be interference near the equipment marked with the following symbol: 
NOTE 1 At 80MHz and 800MHz, the highest frequency range applies. NOTE 2 These directives may not be applicable in every situation. The electromagnetic transmission is affected by the absorption and reflection of structures, objects and people. ^a The field intensities set by the fixed transmitters, such as radio base stations, telephones (mobile phone, wireless) land mobile radio, amateur radio, AM and FM radio transmissions and TV transmissions can not be predicted with accuracy. Due to the RF fixed transmitters is recommended to install an electromagnetic inspection at the local in order to evaluate the electromagnetic environment. If at the place where the equipment is be using the field intensity level exceeds the conformity level for the RF above, is recommended to observe if the operations are normal. Whether abnormal operations are observed, additional procedures shall be necessary such as reorientation or replace the equipment. ^b Whether above the frequency range of 150kHz to 80 MHz is recommended a field intensity below than 3 V/m.			

TECHNICAL SPECIFICATIONS

Electromagnetic emissions

Guidelines and manufacturer's declaration - electromagnetic immunity			
The Jet Hand I is made to be used in the electromagnetic environments specified below. The client or the user of the Jet Hand I must be sure that it is used in such environment.			
Immunity test	ABNT Test level NBR IEC 60601	Level of compliance	Electromagnetic environment Directives
Electrostatic discharge(ESD) IEC 6100-4-2	± 6 kV Contact ± 8 kV Air	± 6 kV Contact ± 8 kV Air	Floors should be wooden, concrete or ceramic. If the floor is covered with synthetic material, the relative humidity should be at least 30%
Quick electric transitory phases / train of pulses ("Burst") IEC 61000-4-4	± 2 kV in power supply lines ± 1 kV in input / output lines	± 2 kV in power supply lines ± 1 kV in input / output lines	It is advisable that the quality of the power supply should be that of hospital or typical commercial environment
Surges IEC 61000-4-5	± 1 kV lines (s) to lines (s) ± 2 kV lines (s) to ground	± 1 kV lines (s) to lines (s) ± 2 kV lines (s) to ground	It is advisable that the quality of the power supply should be that of hospital or typical commercial environment
Reduction, interruption and variance of voltage in power supply input lines IEC 61000-4-11	$< 5\% U_t$ ($>95\%$ drop in U_t) for 0,5 cycle $40\% U_t$ (60% drop in U_t) for 5 cycles $70\% U_t$ (30% drop in U_t) for 25 cycles $< 5\% U_t$ ($>95\%$ drop in U_t) for 5s	$< 5\% U_t$ ($>95\%$ drop in U_t) for 0,5 cycles $40\% U_t$ (60% drop in U_t) for 5 cycles $70\% U_t$ (30% drop in U_t) for 25 cycles $< 5\% U_t$ ($>95\%$ drop in U_t) for 5s	The recommended power supply quality is the same as used for commercial or hospital environment. If is required a continuous use during energy supply outages, it is recommended that the equipment be feed by an uninterruptible power supply or a battery.
Magnetic field in frequency of power supply (50/60Hz) IEC 61000-4-8	3 A/m	0,3 A/m	If an image distortion occurs, may be necessary place the equipment far from the supply frequency or to install a magnetic armour. The frequency magnetic field shall be measured at the installment place to assure that it is low enough.
NOTE U_t is the a.c. power supply voltage before the application of the test level			

TECHNICAL SPECIFICATIONS

Electromagnetic emissions

Eletromagnetic emissions		
The Jet Hand I is made to be used in the electromagnetic environments specified below. The client or the user of the Jet Hand I must be sure that it is used in such environment.		
Emission test	Compliance	Eletromagnetic environment - Guide
RF emissions ABNT NBR IEC CISPR 11	Group 1	This equipment uses RF energy only for internal functions. However, its emissions are too low and it's unlikely to cause any interference in the equipments next to it.
RF emissions ABNT NBR IEC CISPR 11	Class B	This equipment is proper to be used in all establishments; including domestic settings and those directly connect to a public low voltage distribution which feeds domestic buildings.
Emissions of harmonics IEC 61000-3-2	Class A	
Fluctuation of Voltage / Emissions of flicker IEC 61000-3-3	As per	

Content of accessible and non-accessible demarcations

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MANUFACTURER AND RESPONSIBLE FOR THE WARRANTY

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TECHNICAL SPECIFICATIONS

Packing symbols

It determines the maximum quantity of boxes which can be stacked during transportation and storage "as per packaging".

Packing to be transported and / or stored avoiding humidity, rains and wet floor.

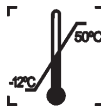
Packing to be transported and / or stored with the harrows up.

The packing must be stored and transported away from direct sun light exposure.

Packing to be transported and / or stored with care (should not suffer drop and neither receive impact).

Temperature limit for the packing to be stored or transported.

Product symbols

Careful: It indicates an important instruction for the operation of the product. Not following it can cause dangerous malfunctioning.

Warning - Consult the manual

Note: It indicates useful information for operation of the product.

B type equipment

Important: It indicates an instruction of safety for operation of the product. Not following it, can lead to serious danger to the patient.

It indicates the appropriate temperature for sterilization in autoclave.

OPERATION OF EQUIPMENT

How to use the Jet Hand I

1) Connect the triple terminal with spray of the delivery unit to the quick coupling located on the back part of the Jet Hand I.

2) Removing the cover (02) add sodium bicarbonate to the container in a sufficient quantity for prophylaxis, i.e. 12g.

3) In order to activate the Jet Hand I it is necessary to remove it from the support and then press the foot control of the delivery unit.

4) In order to activate the Jet Hand I through the delivery unit with the "chip-blower" foot control it is necessary to press the key downwards (A) and the lever to the right (B) together. It will activate the air and water high rotation terminal (spray).

5) Adjust the water outflow through the valve located on the triple terminal with the spray (05).

6) In order to couple or uncouple the bicarbonate Jet handpiece, pull the device from the body of the Jet in the direction indicated (A), and then pull the handpiece (B) in the opposite direction.

ATTENTION: In order to avoid clogging, don't add more than 12g of bicarbonate. Bicarbonate level can be checked through the transparent lid.

Use of the bicarbonate jet

Bicarbonate jet removes dark stains from teeth, caused by tobacco, coffee, tea, etc. related to plaque. In order to obtain better results of its use, we recommend a distance of 5mm between the handpiece and the teeth, with an angle from 30 to 45°, describing circular movements over the teeth.

The bicarbonate jet must be directed to the occlusal edge and not to the gingival edge in order to avoid an unpleasant sensation to the patient.

This equipment shall not be used with patients suffering from respiratory or renal diseases, or under hemodialysis treatment. These cases must be surveyed by a physician.

We recommend the use of safety goggles and mask when using the bicarbonate jet.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Transportation, storage and operation

This equipment must be transported and stored observing the following directions:

- Avoid falls and impacts;
- Keep it dry, do not expose it to rain, water drops or wet floor;
- Keep it away from water and direct sunlight, and in its original wrapping;
- Don't move it over irregular surfaces, protect it from rain and observe the maximum stack quantity specified in the packaging;
- Transportation and storage temperature range: -12°C to 50°C.
- Ambient temperature range recommended by Gnatus +10 ° C to +35 ° C.

The Equipment maintains its condition of safety and efficacy, provided that it is maintained (stored) as mentioned in this instruction of use. Thus, the equipment will not lose or alter its physical and dimensional features.

Sensitivity to environmental conditions in normal situations of use

The equipment has been planned not to be sensitive to interference such as magnetic fields, external electrical factors, electrostatic discharge, pressure or variance of pressure, provided that the equipment is installed, maintained, clean, preserved, transported and operated as per this instruction for use.

Precautions and warnings “during the installation” of equipment

- Install the unit in a place where it will not be damaged by the pressure, temperature, humidity, direct sunlight, dust, salts.
- The unit should not be submitted to inclination, excessive vibrations, or blows (including during transportation and handling).
- This equipment was not planned for use in an environment where vapors, anesthetic mixtures inflammable with air, or oxygen and nitrous oxide can be detected.
- Before the first use and/or after long interruptions from work such as vacations, clean and disinfect the equipment.

These information also make part of the Manual of Installation and Maintenance of the equipment that can be found with the authorized Gnatus technician.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Recommendations for the dental equipment maintenance

Your Gnatus equipment has been designed and developed according to the standards of modern technology. Similarly to other kinds of equipment, it requires special care, which is many times neglected due to several reasons and circumstances.

Therefore, here are some important reminders for your daily routine. Try to follow these simple rules, which will save you a lot of time and will avoid unnecessary expenses once they start making part of your working procedure.

Precautions and warnings “during the use” of equipment

- The equipment should only be operated by duly enabled and trained technicians (Dental Surgeons, Capacitated Professionals)

- If any maintenance should be required, only use services of the Gnatus Authorized Technical Assistance.

- The equipment has been manufactured to handle both continuous and intermittent operation; so follow the cycles described in these Instructions for Use.

- Although this equipment has been planned in accordance with the standards of electromagnetic compatibility, it can, in very extreme conditions, cause interference with other equipment. Do not use this equipment together with other devices very sensitive to interference or with devices which create high electromagnetic disturbance.

- Do not expose the plastic parts to contact with chemical substances, use in the routines of dental treatment, such as: acids, mercury, acrylic liquids, amalgams, etc.

Avoid let sodium bicarbonate in the recipient for long time without being used. The residual air humidity effect will be able to change the properties of the powder and to provoke blockages.

Gnatus shall not be responsible for:

- Use of the equipment differing from that for which it is intended.

- Damages caused to the equipment, the professional and/or the patient by the incorrect installation and erroneous procedures of maintenance, differing from those described in these Instructions for use which come with the equipment or by the incorrect operation of it.

Precautions and warnings “after” the use of equipment

- Always maintain the equipment clean for the next operation.

- Do not modify any part of the equipment. Do not disconnect the cable or other connections without need.

- After using the equipment, clean and disinfect all the parts which may be in contact with the patient.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions and warnings during the “cleaning and disinfection” of equipment

- Before cleaning the equipment, turn off the main switch.
- Avoid spilling water, even accidentally, or other liquids inside the equipment, which could cause short circuits.

CLEANING OF THE EQUIPMENT

Avoid leaving sodium bicarbonate on the reservoir for long periods without using it. The air humidity could change the powder properties and cause a blockage.

- Remove the holding ring from the transparent cover of the bicarbonate container (01);

- Clean the ring, the transparent cover and inside the bicarbonate container;

Attention:

- The ring, transparent cover and container should only be cleaned using flannel or cotton wool moistened with water or the product indicated. Never use any other chemical product to clean these parts.

- Avoid any impact on the cover of the container.
- Do not use the equipment with the cover of the container damaged, as it may break during the application, projecting fragments.
- It is advisable to replace the transparent cover periodically “every 6 months”.

Precautions in case of alteration in the functioning of equipment

- If the equipment has any abnormality, check if the problem is related to any item listed in the topic of unforeseen events (failures, causes and solutions). If it is not possible to resolve the problem, turn off the equipment, remove the power supply cable from the socket and contact your representative (Gnatus).

Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment

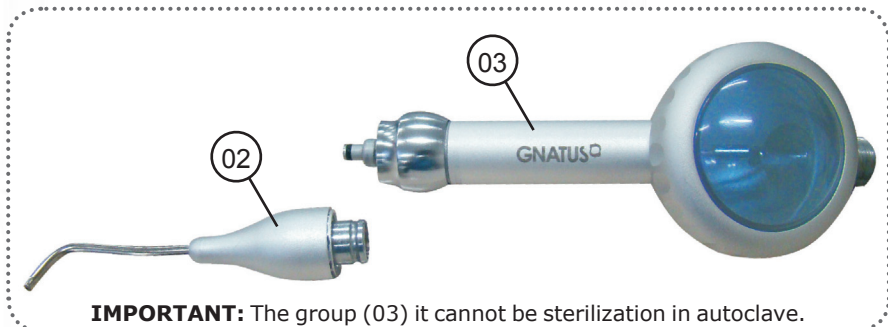
In order to avoid environmental contamination or undue use of the Equipment after it has become useless, it should be discarded in the suitable place (as per the local legislation of the country).

- Pay attention to the local legislation of the country for the conditions of installation and disposal of residue.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Sterilization in Autoclave


Remove the application tip of the bicarbonate jet (02) as per page 37 (item 06) and sterilize it in the autoclave (covered).

IMPORTANT: The group (03) it cannot be sterilization in autoclave.

I Jet Hand features an unpressuring, hose and handpiece internal cleaning automatic system. When pedal is loosed, an internal air jet will clean the system. If the system were clogged, proceed as follows:

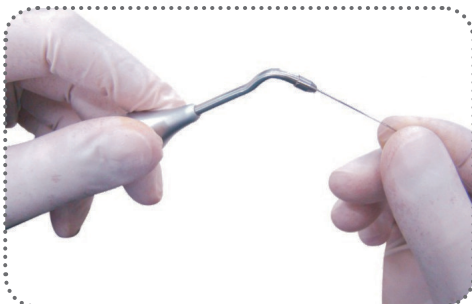- Uncouple the point of the bicarbonate jet;
- Clean the hole with the unblocking device, inserting it several times (03).
- Couple the point again;
- Aim the point of the bicarbonate Jet at a suitable place (water unit, sink basin, etc) and activate the foot control.

Disinfection

- Use clean and soft cloth dampened in alcohol 70% to disinfection of the equipment.
- Never use corrosive disinfectants or solvents.

Note: Use gloves and other systems of protection, during the disinfection.

Cleaning

Important: In order to execute cleaning or any type of maintenance, ensure that the equipment is disconnected from the electrical network.

The cleaning procedure below should be executed at the start of the working day and after each patient.
Always turn off the main switch before executing the procedures of daily maintenance.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

To clean the equipment, we recommend the use of "BactSpray (Reg n° MS: 3.2079.0041.001-5) or any other similar product:

Active component: Benzalkonium chloride (tri-quaternary ammonium)

Solution 50%..... 0.329%

Chemical composition: Butyl Glycol, Decyl polyglucose, Sodium Benzoate, Sodium Nitrate, Essence, Deodorized Propane / Butane, demineralized Water.

For more information concerning cleaning procedures, see manufacturer's instructions.

WARNING:

- In order to prevent risks and damages to equipment, make sure that the liquid does not enter into the unit.
- The application of other solvent-based cleaning products or sodium hypochloride isn't recommended, because they may damage the equipment.

 **NOTE:** The registration at the Ministry of Health of the "BactSpray" is executed separately from the product described in this manual, as the "BactSpray" is not manufactured by Gnatus.

Supply of the Bicarbonate Jet

To clean your equipment, we advise using sodium bicarbonate "Clean Okta" (exempt of registration by the Ministry of Health) or another product with similar features:

Composition of the product: Bicarbonate of Sodium (99,6%), Anidrido Silícico, Essence - 105 spherical standard microns.

Larger information on the product, consult the manufacturer's instructions in the product.

Additional procedures for reuse

The equipment can be reused in undetermined, i.e. unlimited, quantities, only needing to be cleaned and disinfected.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Preventive Maintenance

The equipment should be calibrated routinely, as per the legislation in force in the country. But never with a period exceeding 3 years.

In order to protect your equipment, seek Gnatus technical assistance for periodic revisions of preventive maintenance.

Corrective Maintenance

If the equipment has any abnormality, check if the problem is related to any of the items listed in the item Unforeseen Events (situation, cause and solution).

If it is not possible to solve the problem, turn off the equipment, and request Gnatus technical assistance.

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS

 Upon coming across any problem in operation, follow the instructions below to check and repair the problem, and/or get in touch with your representative.

Problem	Probable cause	Solution
- Not enough bicarbonate in jet	- Shortage of bicarbonate in container. - Container outlet or tip clogged - Too much bicarbonate in the container	- Add bicarbonate to container (max.12g) - Remove clogged parts with auger - Withdraw excess
- Jet low pressure	- Compressor unplugged	- Switch compressor on
- Jet low water	- Regulator Flow (04) closed - Delivery unit water reservoir empty.	- Open the regulator flow - To supply the delivery unit water reservoir with water.
- Leak of bicarbonate.	- Ring of the cover badly placed or damaged.	- To position ring correctly or to substitute.
- Leak of water in the terminal.	- Terminal badly coupled.	- To couple terminal correctly.

WARRANTY OF EQUIPMENT

This equipment is covered by the warranty terms counting from the date of installation, as specified below; provided that the defect has occurred in normal conditions of use and that the equipment has not remained stored for more than 06 months counting from the issue date of the sales document until the date of the actual installation.

- WARRANTY TERMS: Verify the guarantee certificate;

- LOSS OF THE WARRANTY:

- A) Attempt to repair using an inadequate tool or by unauthorized technicians;

- B) Installation of the equipment by an unauthorized technician;

- C) Damage arising from inappropriate storage or signs of infringement;

- D) Incorrect use of the equipment;

- E) Use of a cleaning product not indicated by the factory;

- F) Falls or blows which the equipment may undergo or lack of observation of an compliance with the guidelines of the Owner's Manual, which was delivered with the present document, together with the equipment. Repair or replacement of parts during the warranty period shall not extend the validity term of their warranty.

- This warranty does not exempt the customer from paying the service charge for the visit and the travel expenses of the technician, except when the customer sends the equipment to execute the maintenance inside the establishment of the technical assistance.

- “Consumer Defense Code - art. 50, unique paragraph”.

- The Warranty Certificate comes with the product and must be filled in upon the date of installation by the Gnatus Authorized Technician.

- Queries and information: GNATUS Help Desk (+55) 16 2102-5000.

- Check the warranty term attached to this manual.

FINAL CONSIDERATIONS

The most important aspect related to equipment care is that concerning spare parts.

To guarantee the life span of your equipment, use only **original Gnatus spare parts**. They are sure to follow the technical specifications and standards required by Gnatus.

We must also point out to you our chain of authorized dealers. Only dealers that make part of this chain will be able to keep your equipment constantly new for they count on technical assistants who have been trained and on specific tools for the correct maintenance of your equipment.

Doubts and information: GNATUS Call center (55-16) 2102-5000.

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Obelis S.A, Boulevard Général Wahis 53, 1030 Brussels, Belgium,