PRESENTATION:

Flexible ostby rubber dam frame - Adult;
Flexible ostby rubber dam frame - Kids;
Ostby rubber dam frame simple - Adult;
Ostby rubber dam frame simple - Kids;
Young rubber dam frame;
Packaging with 1 unit.

DESCRIPTION:

Arches are made of sterilizable material that is easy to handle and good resistance, assists in fixing the rubber sheet, providing absolute isolation of the tooth to be treated, preventing contamination.

INDICATION:

Indicated for assist in fixing the rubber sheet.

COMPOSITION:

Polyacetal.

CLEANING AND STERILIZATION PROCEDURE:

1. After using the product, make the removal of organic matter in ultrasonic tubs by using enzymatic soap (minimum 3 enzymes) under dilution recommended by the manufacturer.
2. Rinse under running water and observe the presence of residues, repeat the process or make manual removal.
3. For autoclave sterilization, after cleaning, arrange the parts in the packaging for sterilization and autoclave at 121 °C to 135 °C temperature (250 °F to 275 °F), according to the autoclave manufacturer's recommendations.

DIRECTIONS OF USE:

1. First, make sure the product is clean and sterilized. Stretch the rubber sheet to the maximum extent, thus keeping it for a few seconds so that it provides less tension on the clamp of the arch, preventing release from the tooth with frequency;
2. Attach the rubber sheet on each bow clip;
3. Make sure that the rubber sheet is secure and will not escape;
4. Position the arch with the rubber sheet according to ability and prior

scientific knowledge;

5. After using, perform the cleaning and sterilization procedures. Store on the package itself for sterilization.

WARNINGS AND PRECAUTIONS:

The use is restricted to professionals specialized in dentistry, according to local legislation and with specific training for its handling. Procedures provided in the directions of use should be strictly followed. Use only after the application of cleaning and sterilization procedures. Do not use the product after the expiration of allowed cycles.

CONTRAINDICATIONS AND SIDE EFFECTS:

Patients with sensitivity to polyacetal should not use the product.

STORAGE:

Store in sterilization packaging, at room temperature, away from heat, direct sunlight and humidity. Temperature: below 30 °C. Humidity: less than 75 UR.

TRANSPORT:

Temperature: less than 50 °C. Humidity: below 75 RH. Lighting: keep the product protected from direct sunlight.

DISPOSAL:

The product be disposed of, according to current legislation of its country.

VALIDITY:

Arches have a validity of 50 cycles for sterilization. Do not use after the validity of the permitted cycles. The useful life of this product depends on the care taken during its use. Successive uses and sterilizations cause material fatigue and limit the service life. Check the condition of this product before each use and make the purchase of a new one if excessive fatigue or risk of breakage is observed.

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