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AllPrime allprimedental.com

Autoclavable Cone Radiographic Positioner **EN**

INDICATION:
 The AllPrime Autoclavable Cone Radiographic Positioner helps guide the angle of the dental x-ray machine in periodontal procedures, resulting in standardized and clear radiographic images.

COMPOSITION:
 Thermoplastic, pigment and non-toxic silicone.

PIECES:

- Positioner for upper and lower incisors and canines.
- Positioner for upper right and lower left molars.
- Positioner for upper left and lower right molars. Note: to take lower teeth, the sides of parts 2 and 3 are reversed.
- Positioner for interproximal radiography bite wings.
- Storage pot (non-autoclavable).

MODO DE USO:
How to place the x-ray film:
 Firmly place the radiographic film in the position indicated in the figure, between the holder (for film) and the pressur device.

A – Film support
 B – Pressure device
 C – Bite device
 D – Base of the positioner

Radiographic take:
 The constituents of this product should be used as shown in the illustrations. The X-ray Angle is correctly matched to the position of the film and tube parallel to the positioner.

Storage:
 Store the package in a clean and dry place away from direct sunlight. Temperature between 3 °C to 50 °C and humidity up to 75 RH.

WARNINGS AND PRECAUTIONS:
 Exclusive use of the professional. Do not use the product without prior cleaning and sterilization.

CLEANING/STERILIZATION:
 Before starting the cleaning process, remove the biting device from the parts. Remove organic material using ultrasonic cleaners with enzymatic detergent (minimum 3 enzymes) at dilutions recommended by the detergent manufacturer. Then wash with running water. Pack the pieces in sterilization wrapping (envelopes of surgical grade paper) and put them in the autoclave with temperature from 121 °C to 134 °C, observing the recommendations of the autoclave manufacturer's instruction manual. The pot and cap are not autoclavable, they are only used to store the product after sterilization. Remove the bite device from the positioner to autoclave. Sterilization is valid for one week if the package is not violated. Sterilization can also be performed using, for example, 0.2% peracetic acid solution for 1 hour. Repeat the process after each use.

EXPIRATION DATE:
 Once opened, if stored according to the manufacturer's recommendations, the product remains stable for 5 years after the date of manufacture or for 50 sterilization cycles.

DISPOSAL MODE:
 The product must be disposed of according to the legislation in force in your country.

CONTRAINDICATIONS:
 The product should not be used in cases of known hypersensitivity (allergy) to the product's components.

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