Espelho Metálico

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Metallic Mirror

USE INSTRUCTIONS

METALLIC MIRROR: Allows for perfect image reflection to aid in obtaining intraoral photographs. Indusbello metal mirrors have rounded contours that allow for a comfortable fit in the patient's mouth. During the procedure, the side opposite the one being used will serve as a handle, preventing hands from entering the field of vision.

COMPOSITION: Chrome plated metal.

INDICATION: Mirror for intraoral photography.

Mirror IBM 001: Occlusal – Adult and child;
Mirror IBM 002: Lingual – Adult;
Mirror IBM 003: Side – Adult;
Mirror IBM 004: Occlusal and lateral – Adult;
Mirror IBM 005: Occlusal – Adult and child;
Mirror IBM 006: Occlusal and lateral – Children.

INSTRUCTIONS FOR USE: Choose the mirror according to the region to be photographed. Take the product to the patient's mouth, keeping it in position with the support on the side for use. Carry out the photographic procedure.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. Do not use steel wool sponges or abrasives to clean the mirror. After washing the material, rinse with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection

and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

WARNING: Sterilize before use. Do not use steel wool sponges or abrasives to clean the mirror.

CAUTION: Store the metallic mirror inside the fabric envelope that comes with the product. To prevent metallic mirrors from fogging up during use, direct the air jet at the parts immediately before use and/or when taking the photo. Do not touch the product to the inner walls of the autoclave.

STORAGE: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

EXPIRATION DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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Rev 02 – 18.07.2023 | Cód 326210880003
ANVISA: 80213420017

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Medical device
(Europe).

European Authorized
Representative.

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