Coroas

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Crowns

The TDV Transparent Crowns are produced for deciduous and permanent dentitions, in 3 different sizes, in transparent material and immune to the chemical reaction with the various resins for dental use.

Indication: To use in restorations of compounded photopolymerizable resins, the crowns are indicated to treatment of angles, edges and dental reconstruction. With autopolymerizable resin, it is used to manufacture provisional anterior and posterior teeth.

Use: Select the number and size of the deserved crown. Crowns are identified by the tooth number and its mesial-distal dimension. Adapt the crown, and cut the cervical margin with the help of curved scissor. Fill the crown with the type of desired resin and apply it on the already prepared tooth. Follow the polymerization instructions of the material used. After the polymerization process, remove the crown. The use of interdental wedges and/or other aids depends on the producers and the professional preferences. The crowns can be used with composite, bisacryl or acrylic resin.

Warnings, precautions and restrictions: Professional use only. Single use. Its reuse can cause cross contamination. To avoid an accidental swallowing of this product it's necessary the application of a total isolation technique of the operator field. When there is proven allergy to any of its components, the product must not be used. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Composition: Polyvinyl chlorite

Storage: To be stored in a dry and cool place, protected from sunlight.

Disposal: Dispose in accordance with applicable local requirements.

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Manufacturer/Fabricant: TDV Dental Ltda.

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EC REP Obelis S.A.