

IPS Ivocolor®

Combination table for staining the IPS veneering materials and the all-ceramic materials of the IPS e.max® system*

Kombinationstabelle für die Bemalung von IPS Verblendkeramiken und Vollkeramiken des IPS e.max® Systems*

		A-D tooth shade Zahnfarbe																			
		BL1	BL2	BL3	BL4	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
IPS Ivocolor Shade	Shade 0																				
	Shade 1																				
	Shade 2																				
	Shade 3																				
	Shade 4																				
	Shade 5																				
	Shade 6																				
	Shade 7																				
	Shade 8																				

* Other shade combinations apply when using IPS e.max ZirCAD LT discs in the LT group shades.

Bei der Verwendung von IPS e.max ZirCAD LT Discs in den LT-Gruppenfarben gelten andere Farbkombinationen.



IPS Ivocolor is not suitable for the staining, characterization and glazing of partially crystallized IPS e.max CAD restorations (in the blue state).

IPS Ivocolor ist nicht geeignet für die Bemalung und Glasur von teilkristallisierten IPS e.max CAD-Restaurationen (im blauen Zustand).

IPS Ivocolor®

Combination table for IPS Ivocolor® components

Kombinationstabelle für IPS Ivocolor® Komponenten

Components Komponenten	Initial mixing Erstmaliges Anmischen			Refreshing of dried-out materials Auffrischung von eingetrockneten Massen		
	IPS Ivocolor Mixing Liquid allround 	IPS Ivocolor Mixing Liquid longlife 	IPS Ivocolor Essence Fluid 	IPS Ivocolor Mixing Liquid allround 	IPS Ivocolor Mixing Liquid longlife 	IPS Ivocolor Essence Fluid 
 IPS Ivocolor Essence	✓	✓	✓	✓	✓	—
 IPS Ivocolor Shade	✓	✓	—	✓	✓	—
 IPS Ivocolor Glaze Powder/ FLUO	✓	✓	—	✓	✓	—
 IPS Ivocolor Glaze Paste/ FLUO	✓	✓	—	✓	✓	—

Rx only

CE 0123



Manufacturer
Ivoclar Vivadent AG, Bendererstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com



See Instructions

Date information prepared: 2022-06, Rev. 2
668431

ivoclar