GC AMERICA INC.
 3737 W 127th STREET
 ALSIP, ILLINOIS 60803
 TEL (708) 597-0900
 FAX (708) 597-4031
 www.gcamerica.com

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A QUEM POSSA INTERESSAR:

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Atenciosamente,
 GC AMERICA INC.

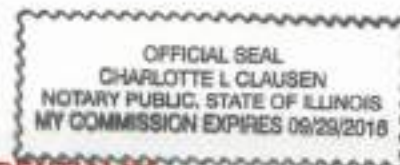
Mark A. Heiss, DDS
 Director of Regulatory Affairs

SUBSCRIBED AND SWORN TO

before me this 27
 day of March, 2014

Charlotte L. Clausen
 Notary Public

Illinois Will
 State, County

PLEASE TREAT THIS PROPRIETARY INFORMATION AS CONFIDENTIAL

GC Fuji IX GP™ FAST

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

GC Fuji IX GP FAST is a fast-setting version of GC Fuji IX GP.

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RECOMMENDED INDICATIONS

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Core build-up.

CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

| | Fast | Original |
|--|--------------------|-----------|
| Powder / Liquid Ratio (g/g) | 0.40/0.11 | 0.40/0.11 |
| Mixing Time (sec.)* | 10" | 10" |
| Time of initial extrusion after mixing | 10 seconds maximum | |
| Working Time (min., sec.)* | 1'15" | 2'00" |
| Net Setting Time (min., sec.)* | 2'00" | 2'20" |
| Final Finishing Commencing Time (37°C, after starting mix) (min.,sec.) | 3'00" | 6'00" |

Test conditions : Temperature (23 +/-1°C), Relative humidity (50 +/-5%)

1. CAVITY PREPARATION

- Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.
- Apply GC CAVITY CONDITIONER (10 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).
- Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

2. CAPSULE ACTIVATION AND MIXING

- Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).
- To activate the capsule, push the plunger until it flush with the main body and hold it down for 2 seconds (Fig. 3).
Note :
1) Ensure the plunger is fully pressed to avoid the incorrect mixing ratio of powder and liquid.
2) The capsule should be activated just before mixing and used immediately.
- Immediately set it into a mixer (or an amalgamator) and mix for 10 seconds (~ 4,000RPM) (Fig. 4).

3. RESTORATIVE TECHNIQUE

- Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.
- Make two clicks to prime the capsule then syringe (Fig. 5). The working time is 1 minute 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 6).
- Form the preliminary contour, and cover with a matrix if required.
- During the first 3 minutes from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply GC Fuji Varnish (blow dry) or GC Fuji Coat LC (light cure).
Note :
1) To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
2) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.

4. FINISHING

- Final finishing under water spray using standard techniques can begin approximately 3 minutes after start of mixing (Fig. 7).
- Apply a final coat of GC G-COAT Plus and light cure for 20 seconds (Fig. 8,9). Alternatively, apply GC Fuji Varnish (blow dry) or GC Fuji Coat LC (light cure) Refer to the respective instructions for use.
- Instruct the patient not to apply pressure for 1 hour.

STORAGE

Recommended for optimal performance, store the capsules in the original aluminum foil in a cool and dark place (4-25°C)(39.2-77.0°F).

SHADES

A1, A2, A3, A3.5, B2, B3 and C4.
Shade numbers according to Vita® shade guide.
Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

- Single shade package : 48 capsules.
- Assortment package : 48 capsules (10 each of A2, A3 and A3.5; 9 each of B2 and B3).
Average contents per capsule : 0.40g powder and 0.11g (0.09mL) liquid.
Minimum net volume of mixed cement per capsule : 0.14mL.
Option : GC Capsule Applier (1 piece).
- Single Shade Export package: 200 capsules.

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available for the Americas at: <http://www.gcamerica.com>

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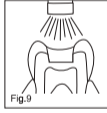
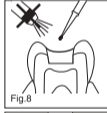
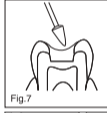
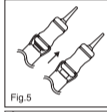
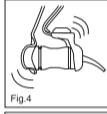
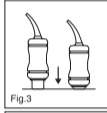
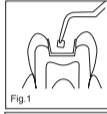
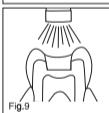
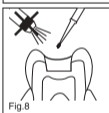
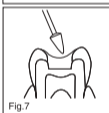
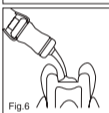
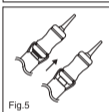
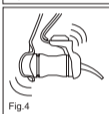
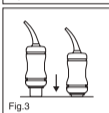
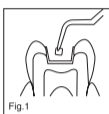
| | |
|--|-----------|
| Prior to use, carefully read the instructions for use. | EN |
|--|-----------|

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RECOMMENDED INDICATIONS

As a base or liner in prepared cavities.

CONTRAINDICATIONS

- Pulp capping.
- Avoid use of this product in patients with known glass ionomer cement allergies.

DIRECTIONS FOR USE

- POWDER AND LIQUID DISPENSING
 - The standard powder to liquid ratio is 1.2 g/1.0 g. 1 level scoop of powder to 1 drop of liquid.

Note:

For a base, a heavier consistency is desirable. Compress powder in the scoop before levelling.
 - For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
 - Hold the liquid bottle vertically and squeeze gently.
 - Close bottles immediately after use.
- MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into two equal parts. Mix the first portion with all of liquid for 5-10 seconds.

Incorporate the remaining portion and mix the whole thoroughly for 10-15 seconds.
- RESTORATIVE TECHNIQUE
 - Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
 - Wash and dry but **DO NOT DESICCATE**. CAVITY CONDITIONER or DENTIN CONDITIONER is recommended to remove the smear layer.
 - Mix the required amount of cement.
 - Remove surface moisture, but **DO NOT DESICCATE**.
 - Transfer cement to the preparation using a syringe or suitable placement instrument, covering dentine up to the dentino-enamel junction.
 - Allow cement to set (about 4 minutes).
 - Proceed with standard composite or amalgam restorative techniques.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

PACKAGES

- Bottle of 15 g powder with measure
- Bottle of 15 g (12 mL) liquid
- 1-1 pkg. : 15 g powder, 15 g (12 mL) liquid, powder spoon, plastic spatula, mixing pads

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT** mix powder or liquid with any other glass ionomer components.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. **DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gcamerica.com> They can also be obtained from your supplier.

Last revised: 12/2019

| |
|--|
|  |
| <p>MANUFACTURED by</p> <p>GC CORPORATION</p> <p>76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan</p> |
| <p> EU : GC EUROPE N.V.</p> <p>Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00</p> |
| <p>RESPONSIBLE MANUFACTURER IN CANADA:</p> <p>GC AMERICA INC.</p> <p>3737 West 127th Street, Alsip, IL 60803 U.S.A.</p> |
| <p>DISTRIBUTED by</p> <p>GC CORPORATION</p> <p>76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan</p> |
| <p>GC AMERICA INC.</p> <p>3737 West 127th Street, Alsip, IL 60803 U.S.A.</p> <p>TEL: +1 708-937-0900</p> <p>www.gcamerica.com</p> |
| <p>GC SOUTH AMERICA</p> <p>Rua Heliódora, 399, Santana - São Paulo, SP, BRASIL</p> <p>CEP: 02022-051 - TEL: +55-11-2925-0365</p> <p>CNPJ: 08.279.999/0001-61</p> <p>RESP. TÊC: Mayara de Santis Ribeiro - CRO/SP 105.982</p> |
| <p>GC ASIA DENTAL PTE. LTD.</p> <p>5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541</p> <p>TEL: +65 6546 7589</p> |
| <p>GC AUSTRALASIA DENTAL PTY. LTD.</p> <p>1753 Botany Rd, Banksmeadow, NSW 2019, Australia</p> <p>TEL: +61 2 9301 8200</p> |
| <p>PRINTED IN JAPAN</p> |

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RECOMMENDED INDICATIONS

CAVITY CONDITIONER is a mild polyacrylic acid solution designed to remove the smear layer and to condition dentine and enamel, thus enhancing the bond between glass ionomer cement and tooth structure.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- After tooth preparation, apply CAVITY CONDITIONER to the bonding surfaces for 10 seconds using a cotton pellet or sponge.
- Rinse thoroughly with water and dry. DO NOT DESICCATE.
- Apply GC Fuji BOND LC, GC Fuji II LC (improved) or other glass ionomer as a bonding agent, base/liner or restorative material.
- Close the bottle immediately after use.

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Bottle of 6g (5.7mL).

CAUTION

- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

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BULK FILLED RADIOPAQUE POSTERIOR RESTORATIVE IN CAPSULES

EQUIA® Coat

LIGHT CURED SELF ADHESIVE WEAR RESISTANT COATING

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RECOMMENDED INDICATIONS

1. Class I restorations
2. Non-stress bearing Class II restorations
3. Stress bearing Class II restorations when isthmus is less than half of interspal distance
4. Intermediate restorative
5. Class V and root surface restorations
6. Core build up

EQUIA Coat is used to seal and protect the surface of EQUIA Fil restorations.

CONTRAINDICATIONS

1. Pulp capping.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

NOT TO BE USED

EQUIA Coat should not be used in combination with desensitizers and eugenol containing materials as these may hinder EQUIA Coat from setting or bonding properly.

DIRECTIONS FOR USE

| | |
|--|--------------------|
| Powder / Liquid Ratio (g/g) | 0.40 / 0.12 |
| Mixing Time (sec.) | 10" |
| Time of initial extrusion after mixing | 10 seconds maximum |
| Working Time (min., sec.) | 1'15" |
| Net Setting Time (min., sec.) | 2'00" |
| Final Finishing Commencing Time (37°C, after starting mix) (min.,sec.) | 2'30" |
| VLC curing time of EQUIA Coat | 20" |

Test conditions: Temperature (23 +/-1°C), Relative humidity (50 +/-5%)

1. CAVITY PREPARATION

- a) Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.
- b) Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.
- c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

2. CAPSULE ACTIVATION AND MIXING

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder.
- b) To activate the capsule, push the plunger until it is flush with the main body.
- c) Immediately place the capsule into a metal GC CAPSULE APPLIER and click the lever once. The capsule is now activated. Note: The capsule should be activated just before mixing and used immediately.
- d) Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/-4,000RPM).

3. RESTORATIVE TECHNIQUE

- a) Immediately remove the mixed capsule from the mixer and load it into the GC CAPSULE APPLIER.
- b) Make two clicks to prime the capsule then syringe. The working time is 1 minute 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- c) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation.
- d) Form the preliminary contour, and cover with a matrix if required.
- e) During the first 2 minutes 30 seconds from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply EQUIA Coat and light cure. Note: 1) To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body. 2) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.

4. FINISHING

Finish under water spray using superfine diamond burs after 2 minutes 30 seconds from start of mixing. Note: When applying EQUIA Coat on the existing EQUIA restorations, roughen the surface to be coated with superfine diamond burs.

5. COATING

- a) Spray preparation dust away with water. Dry by gently blowing with oil free air. Surfaces to be coated should be dry. Do not desiccate.
- b) Dispense a few drops of EQUIA Coat into a disposable dispensing dish. Replace bottle cap immediately after use.
- c) IMMEDIATELY apply (within 1 minute after dispensing) to the surfaces to be coated using a micro-tip applicator. Make sure that a disposable micro-tip applicator is firmly fixed on the applicator holder. Use floss to apply to approximal surface. DO NOT AIR BLOW. Note: 1) Non bur cut enamel should be treated with a 35-40% phosphoric acid according to the manufacturer's instructions. Rinse and dry with oil free air. 2) Should the coated surface be contaminated with water, blood or saliva prior to light curing, wash and dry the surface and repeat the coating procedure.

6. LIGHT CURING

IMMEDIATELY light cure all coated surfaces with a visible light curing unit (> 500mW/cm² : Halogen, GC G-Light, LED) for 20 seconds. Note: 1) Place light source as close as possible to the coated surface. 2) If surface is tacky or yellowish, repeat light curing. 3) Use a protective light shield or similar protective eye wear during light curing.

STORAGE

Recommended for optimal performance, store the EQUIA Fil capsules in the original aluminum foil in a cool and dark place (4-25°C)(39,2-77,0°F). If not in use for a prolonged period of time, store EQUIA Coat in refrigerator, otherwise store at room temperature (4-25°C)(39,2-77,0°F).

Shelf life: EQUIA Fil - 2 years from date of manufacture EQUIA Coat - 3 years from date of manufacture

SHADES

A1, A2, A3, A3.5, B1, B2, B3 and C4. Shade numbers according to Vita® shade guide. Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

1. Single shade package: 48 capsules.
2. Assortment package: 48 capsules (10 each of A2, A3 and A3.5; 9 each of B1 and B3). Average contents per capsule: 0.40g powder and 0.12g (0.10mL) liquid. Minimum net volume of mixed cement per capsule : 0.14mL.
3. EQUIA Coat 4mL (1), Disposable Dispensing Dish (20), Micro Tip Applicator (48), Applicator Holder Option: GC CAPSULE APPLIER (1 piece).

CAUTION

1. In case of contact with oral tissue or skin, immediately remove with a sponge or cotton pellet. After the restorative treatment is finished, rinse thoroughly with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. EQUIA Coat is flammable. Do not use near naked flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
4. EQUIA Coat is volatile. Use in a well ventilated place. Replace cap immediately.
5. If the tissue contacted by EQUIA Coat turns white or forms a blister, advise the patient to leave the affected area undisturbed, until the mark disappears, usually in 1-2 weeks. To avoid contact, it is recommended to apply cocoa butter to the area where rubber dam cannot cover.
6. Do not use EQUIA Coat in combination with desensitizers and eugenol containing materials as these may hinder EQUIA Coat from setting or bonding properly.
7. Avoid inhalation or ingestion of material.
8. Avoid getting material on clothing
9. Do not mix with other products.
10. Instruct the patient not to apply pressure for 1 hour.

Last revised: 03/2014

MANUFACTURED by GC CORPORATION 76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by GC AMERICA INC. 3737 West 127th Street, Alsip, IL 60803 U.S.A. TEL: +1-708-587-0900

PRINTED IN THE U.S.A.

Prior to use, carefully read the instructions for use.

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RECOMMENDED INDICATIONS

- Restoration of primary teeth.
- Core build-up.
- Restorations of Class III, V and limited Class I cavities.

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DIRECTIONS FOR USE

- POWDER AND LIQUID DISPENSING
 - The standard powder to liquid ratio is 2.7/1.0g,1 level scoop of powder to 1 drop of liquid.
 - For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
 - Hold the liquid bottle vertically and squeeze gently.
 - Close bottles immediately after use.
- MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all of the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (total time within 30 seconds);
- RESTORATIVE TECHNIQUE
 - Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
 - Wash and dry but **DO NOT DESICCATE**. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear layer.
 - Mix the required amount of cement. Working time is 1 minute 45 seconds at 23°C (73.4°F). Higher temperatures will shorten working time.
 - Remove surface moisture, but **DO NOT DESICCATE**.
 - Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
 - Form the contour, and place a matrix if required during the first 2 minutes of setting.
 - Setting time is 5’30” from start of mixing. Once set, remove the matrix and immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure).
- FINISHING
 - One visit treatment
 - After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
 - Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
 - Final finishing and polishing under water spray can now be carried out.
 - Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.).
 - Standard method
 - 24 hours after placement finish and polish the restoration under water spray using standard techniques.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 3 years from date of manufacture).

SHADE

No. 21 - pale yellow
No. 22 - yellow brown
No. 23 - dark grey
No. 26 - deep brown
M - Modifier

PACKAGES

- Bottle of 15g powder with measure.
- Bottle of 10g liquid (8.0 mL).
- 1-1 pkg. : 15g powder, 10g (8.0mL) liquid, powder measure, mixing pad (No. 22), ISO 9917 (Dental water based cements)

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DIRECTIONS FOR USE

| | |
|-------------------------------|-----------|
| Powder / Liquid Ratio (g/g) | 3.6 / 1.0 |
| Mixing Time (sec.) | 25-30* |
| Working Time (min., sec.) | 2'00" |
| Net Setting Time (min., sec.) | 2'20" |

Test conditions : Temperature (23 +/-1°C)
Relative humidity (50 +/-5%)

1. POWDER AND LIQUID DISPENSING

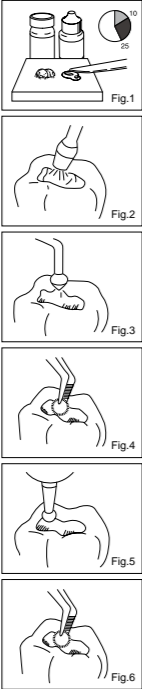
- The standard powder to liquid ratio is 3.6g/1.0g. (1 level scoop of powder to 1 drop of liquid).
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Close bottles tightly immediately after use.

2. MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (Fig. 1).

3. RESTORATIVE TECHNIQUE

- Prepare the tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.
 - Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.
 - Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening) (Fig. 2).
 - Mix the required amount of GC Fuji IX GP. Working time is 2 minutes from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
 - Transfer cement to the preparation using a syringe or other suitable instrument. Avoid air bubbles.
 - Form the preliminary contour and cover with a matrix if required (Fig. 3).
 - When set, immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) (Fig. 4).
4. FINISHING
- Final finishing under water spray using standard techniques can begin at 6 minutes after starting the mix (Fig. 5).
 - Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.) (Fig. 6).
 - Instruct the patient not to apply pressure for 1 hour.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life : 3 years from date of manufacture).

SHADES

A2, A3, A3.5, B2, B3, C4
Shade numbers according to Vita® shade guide.
®Vita is a trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

- 1-1 package : 15g powder, 8g (6.4mL) liquid, powder scoop, mixing pad (No. 22), plastic spatula.
- Bottle of 15g powder with scoop.
- Bottle of 8g (6.4mL) liquid.

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.

Last revised : 02/2014

GC

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1 708-937-0900

GC ASIA DENTAL PTE. LTD.
11 Tampines Concourse, #03-05, Singapore 528729
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61 2 9316 4499

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RECOMMENDED INDICATIONS

1. Cementation of metal or porcelain fused to metal crowns, bridges, inlays.
2. Cementation of stainless steel crowns or orthodontic appliances retained with stainless steel crowns.
3. Cementation of orthodontic bands.
4. Base or liner.

CONTRAINDICATIONS

1. Pulp capping.
2. In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

| Powder/liquid ratio (g/g) | 1.4/1.0 | 1.8/1.0 | 2.2/1.0 |
|------------------------------|---------|---------|---------|
| Mixing time (min. sec.) | 20" | 20" | 20" |
| Working time (min. sec.) | 2'30" | 2'00" | 1'15" |
| Net setting time (min. sec.) | 2'50" | 2'30" | 2'20" |

Test conditions : temperature (23±1°C)
Relative humidity (50±10%)
ISO 9917 : 1991 (E) (Dental water based cements)

1. POWDER AND LIQUID DISPENSING

- a) The standard powder to liquid ratio is 1.8 / 1.0g. 1 level scoop of powder to 2 drops of liquid.
- b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- c) Hold the liquid bottle vertically and squeeze gently.
- d) Close bottles immediately after use.

2. MIXING

Dispense powder and liquid onto the pad. Using a plastic spatula, add all the powder to the liquid. Mix rapidly for 20 seconds.

Note :

For larger amounts, divide the powder into two equal parts. Mix the first portion with all of liquid for 5 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15 seconds (total 20 seconds).

3. CEMENTATION TECHNIQUE

- a) Prepare tooth using standard techniques. For pulp capping use calcium hydroxide.
- b) Clean with pumice and water only. Wash and dry but DO NOT DESICCATE. DO NOT REMOVE SMEAR LAYER.
- c) Clean and dry inside of casting or stainless steel crown.
- d) Mix the required amount of cement.
- e)
 - (i) Cementation of cast metal crowns or bridges
Coat internal surface of restoration with 1mm of cement and start to seat within 30 seconds of completing mix. (DO NOT OVERFILL). Higher temperatures will shorten working time.
 - (ii) Cementation of stainless steel crowns
Fill the adapted crown with cement and seat.
Note :
To facilitate removal of orthodontic appliances, coat the occlusal surfaces of the teeth to be cemented with a fluoride gel or petroleum jelly prior to cementation.
- f) Setting time is 4 minutes 30 seconds after start of mixing. Remove excess cement at the first formation of gel stage.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life : 3 years)

SHADE

Light yellow.

PACKAGES

1-1 package : 15g powder, 10g (8mL) liquid, powder scoop.

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DISTRIBUTED by
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.

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DIRECTIONS FOR USE

| | |
|-------------------------------|-----------|
| Powder / Liquid Ratio (g/g) | 3.6 / 1.0 |
| Mixing Time (sec.) | 25-30" |
| Working Time (min., sec.) | 2'00" |
| Net Setting Time (min., sec.) | 2'20" |

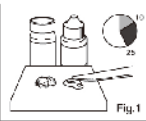
Test conditions : Temperature (23±1°C), Relative humidity (50±10%)
ISO 9917-1 : 2003(E) (Glass polyalkenoate cement)

1. POWDER AND LIQUID DISPENSING

- The standard powder to liquid ratio is 3.6g/1.0g (1 level scoop of powder to 1 drop of liquid).
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Close bottles tightly immediately after use.

2. MIXING

- Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all the liquid for 10 seconds.

- Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (Fig. 1).

3. RESTORATIVE TECHNIQUE

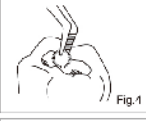
- Prepare the tooth using standard techniques.

Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.


- Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge.


- Rinse thoroughly with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe. DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening) (Fig. 2).

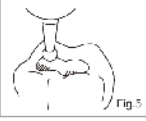
- Mix the required amount of cement. Working time is 2 minutes from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.

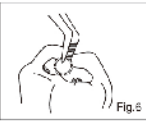
- Transfer cement to the preparation using a syringe or other suitable instrument. Avoid air bubbles.

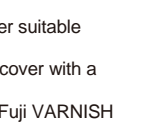- Form the preliminary contour and cover with a matrix if required (Fig. 3).

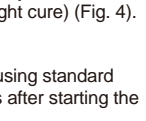- When set, immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) (Fig. 4).

4. FINISHING

- Final finishing under water spray using standard techniques can begin at 6 minutes after starting the mix (Fig. 5).
- Apply a final coat of GC Fuji VARNISH or GC Fuji COAT LC (Fig. 6).
- Instruct the patient not to apply pressure for 1 hour.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life : 3 years from date of manufacture)

SHADES

A2, A3, A3.5, B2, B3, C4

Shade numbers according to Vita® shade guide.

® Vita is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

PACKAGES

- 1-1 Package: 15g powder, 8g (6.4mL) liquid, powder scoop, mixing pad (No. 22)
- Bottle of 15g powder with scoop.
- Bottle of 8g (6.4mL) liquid.
- Mini pack*: 5g powder, 3g (2.4mL) liquid, powder scoop

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GC

MANUFACTURED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL : +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL : +65 6546 7588

PRINTED IN JAPAN

160273SK

GC Fuji TRIAGE™

RADIOPAQUE GLASS IONOMER PROTECTION AND TEMPORARY RESTORATIVE MATERIAL

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Fissure protection.
2. Root surface protection.
3. Hypersensitivity prevention and control.
4. Protection for immature enamel.
5. Temporary filling of endodontic access.
6. Intermediate restorations.

CONTRAINDICATIONS

1. Pulp capping.
2. Avoid use of this product in patients with known glass ionomer cement allergies.

DIRECTIONS FOR USE
PINK Shade is command set, WHITE Shade is chemical set only.

| | |
|--|-----------|
| Powder / Liquid Ratio (g / g) | 1.8 / 1.0 |
| Mixing Time (sec.) | 20-25* |
| Working Time (min., sec.) | 1' 40" |
| Net Setting Time (min., sec.) | 2' 30" |
| Final Finishing Comming Time | 6'00" |
| Final Finishing Comming Time if light cured (PINK Command set) | 4'00" |

Test conditions : Temperature (23±1°C),
 Relative humidity (50±5%)

A. FISSURE PROTECTION

ROOT SURFACE PROTECTION
 HYPERSENSITIVITY PREVENTION AND CONTROL
 PROTECTION FOR IMMATURE ENAMEL

1. Preparation of the tooth surfaces (e.g., for fissure protection or root surface protection).

a) After cleaning the tooth surfaces (prophylaxis with pumice and water) in usual manner, rinse thoroughly with water. Avoid aggravating the operculatum.

Note:
 If extra retention is desired, application of CAVITY CONDITIONER (10 seconds) or DENTIN CONDITIONER (20 seconds) is recommended.

The conditioner should be rinsed off thoroughly after application.

b) Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. A-2). DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Powder and Liquid Dispensing

a) The standard powder to liquid ratio is 1.8g / 1.0g. (1 level scoop of powder to 1 drop of liquid).

b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.

c) Make sure the liquid nozzle is clean and dry before dispensing liquid. Turn the liquid bottle horizontal and hold in this position briefly to remove any bubbles.

d) Then invert and hold the liquid bottle vertically and squeeze gently to dispense a bubble free drop of liquid. After dispensing, wipe any residual liquid off the nozzle.

e) Close bottle caps tightly immediately after use.

3. Mixing
 Dispense the required amounts of powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Spread the liquid over the pad (Fig. A-2) and mix the first portion with all the liquid for 10 seconds. Incorporate the remaining portion and mix the whole amount thoroughly for 15 seconds (total time within 25 seconds).

Working time is 1 minutes 40 seconds from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.

a) Take the mixed material using a suitable placement instrument or brush and apply to the tooth surface (Fig. A-3). Then use a bristly brush to spread thoroughly.

b) Apply to the root surface or occlusal surface and into the pits and fissures.

Note:
 If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

c) After placement, when the material starts to lose the glossy appearance or after curing with the light curing device, apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) to the sealed area and the margins using a cotton pellet or sponge (Fig. A-4).

d) Rinsing under air water spray can be performed 6 minutes from start of mix (chemically set) or 4 minutes if light cured. Use a superfine diamond bur or a silicone finishing point (Fig. A-5).

d) Apply GC Fuji VARNISH or GC Fuji COAT LC to the area again (Fig. A-6).

B. TEMPORARY FILLING OF ENDODONTIC ACCESS

1. Cleaning the root surface

a) After appropriate pulp treatment, clean and gently dry the pulp chamber with an air syringe (Fig. B-1).

b) Fill the chamber with a cotton pellet (Fig. B-2).

2. Powder / Liquid Dispensing and Mixing

See directions in A (above), Section 2 and 3.

3. Placement

a) Using a suitable instrument, place the mixed material over the cotton pellet (Fig. B-3).

Note:
 If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

b) Moisture protection and Finishing
 See directions in A, Section 4, b, c, and d.

C. INTERMEDIA RESTAURATIONS

1. Caries removal

a) Remove caries and loose debris with hand instruments.

b) For better retention, it is recommended to gently clean the carious surface with CAVITY CONDITIONER for 10 seconds or DENTIN CONDITIONER for 20 seconds.

c) Rinse chamber with water. Dry by blotting with a cotton pellet or gently blowing with an air syringe (Fig. C-1). DO NOT DESICCATE. Best results are obtained when prepared surfaces appear moist (glistening).

2. Powder / Liquid Dispensing and Mixing

See directions in A (above), Section 2 and 3.

3. Placement

a) Take the mixed material using a suitable placement instrument or brush and apply to the prepared lesion or tooth surface without incorporating any bubbles (Fig. C-2). Form the contour and if possible cover with a matrix.

Note:
 If a faster set is desired, use a visible light curing device* for 20-40 seconds. Place light source as closely as possible to the cement surface. This function applies only to the Pink Shade. After light cure, it is advisable to protect the surface with a varnish.

b) Moisture protection and Finishing
 See directions in A, Section 4, b, c, and d.

* NOTE:
 The initial set of GC Fuji TRIAGE PINK can be accelerated using the energy of a dental halogen light curing device.

SHADE

Pink, White

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39-77.0°F).

PACKAGE

15g powder, 10g (8.0 mL) liquid, Dentin Conditioner 6g (5.7 mL), powder scoop, mixing pad, 22, plastic spatula

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. DO NOT mix powder or liquid with any other glass ionomer components.

4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

5. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<http://www.georgepoc.com>

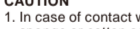
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects- Reporting:
 If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
 as well as to our internal equipment system: vigilance@gc.dental
 In this way you will contribute to improve the safety of this product.

Last revised: 07/2019

MANUFACTURED BY:
 GC CORPORATION
 751 Higashi-Araji, Himeji, Japan, Tokyo 174-8605, Japan

DISTRIBUTED BY:
 GC CORPORATION
 751 Higashi-Araji, Himeji, Japan, Tokyo 174-8605, Japan

EU: GC EUROPE N.V.
 Leuvensesteenweg 13
 B-3001 Leuven, Belgium Tel: +32 16 74 10 00

PRINTED IN JAPAN

Prior to use, carefully read the instructions for use. **EN**

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ENGLISH

GC Gold Label Universal Restorative

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RECOMMENDED INDICATIONS

1. Restoration of primary teeth.
2. Core build-up.
3. Restorations of Class III, V and limited Class I cavities.

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DIRECTIONS FOR USE

| | |
|------------------------------|-----------|
| Powder / Liquid ratio (g/g) | 2.7 / 1.0 |
| Mixing time (sec.) | 25"-30" |
| Working time (min. sec.) | 2'00" |
| Net setting time (min. sec.) | 2'20" |

Test conditions : Temperature (23+/-1°C), Relative humidity (50+/-10%) ISO 9917-1 : 2003(E) (Glass Polyalkenoate Cement)

1. POWDER AND LIQUID DISPENSING

- a) The standard powder to liquid ratio is 2.7/1.0g. 1 level scoop of powder to 1 drop of liquid.
- b) For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- c) Hold the liquid bottle vertically and squeeze gently.
- d) Close bottles immediately after use.

2. MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all of the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (total time within 30 seconds).

3. RESTORATIVE TECHNIQUE

- a) Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping use calcium hydroxide.
- b) Wash and dry but DO NOT DESICCATE. GC CAVITY CONDITIONER or GC DENTIN CONDITIONER is recommended to remove the smear layer.
- c) Mix the required amount of cement. Working time is 2 minutes from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
- d) Remove surface moisture, but DO NOT DESICCATE.
- e) Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
- f) Form the contour, and place a matrix if required during the first 2 minutes of setting.
- g) Setting time is 5 minutes 30 seconds after start of mixing. Once set, remove the matrix and immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (Light Cure).

4. FINISHING

- a) One visit treatment :
 - 1) After initial set, correct the contour WITHOUT WATER SPRAY using standard techniques.
 - 2) Apply GC Fuji VARNISH or GC Fuji COAT LC and wait for about 15 minutes.
 - 3) Final finishing and polishing under water spray can now be carried out.
 - 4) Apply a final coat of GC Fuji VARNISH or GC Fuji COAT LC.
- b) Standard method :
24 hours after placement finish and polish the restoration under water spray using standard techniques.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 3 years from date of manufacture)

SHADE

No. 21 - pale yellow
No. 22 - yellow brown
No. 23 - dark grey
No. 26 - deep brown

PACKAGES

1. 1-1 Package: 15g powder, 10g (8.0mL) liquid, powder scoop, mixing pad (No. 22)
2. Bottle of 15g powder with scoop.
3. Bottle of 10g (8.0mL) liquid.
4. Mini pack*: 10g powder, 7g (5.6mL) liquid, powder scoop

* Mini pack is only available in limited shades (No. 21, 22).

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. DO NOT mix powder or liquid with any other glass ionomer components.

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De kleuren corresponderen met de Vita® kleurenring. © Vita is een gedeponeerd handelsmerk van de Vita Zahnfabrik, Bad Säckingen, Duitsland