Content:

Bottle with 100 mL.

Indication:

Suitable for disinfection of cavitary preparations, crown preparations, inlays, antiseptics of professional's and assistant's hands, for extra-oral antiseptics in areas that will go through a surgical intervention, surfaces and equipment.

Description of the Product:

The CLORHEXIDINE SOLUTION 2% is an antiseptic with antibacterial activity which is effective against both gram-positive and gram-negative bacteria (including Pseudomonas), fungi, yeast and some dermatophylus.

How to use:**Cavities:**

1. With help of a piece of cotton, brush or micro applier, clean the indicated area. 2. Remove the excess and dry the cavity with air jet.

Surfaces:

Use the "spray-wipe-spray" technique.

Instruments:

1. With a piece of cotton soaked in the solution, clean the instrument carefully. 2. Then, dry it with the air jet. If necessary, dip it into the solution for minute and dry it again.

Precautions:

Product of professional use only; Avoid ingesting the product; Avoid contact of the product with the tongue or oral region; Do not use the product for rinsing the mouth, for in this concentration it might cause extrinsic tooth patch, tongue and restoration stains, burning mouth and taste alterations, although it is a reversible process when ceasing the treatment; Do not use the product if its shelf life is expired. Do not use it in patients with sensitivity to Clorhexidine.

Composition:

Chlorhexidine digluconate 2%, metilparabene, purified water.

Storage and transportation conditions:

Store in a clean, dry place, protected from direct sunlight and at a minimum temperature of 5 °C and maximum of 40 °C.

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MADE IN BRAZIL

maquira[®]
Dental Group**Fabricante | Manufacturer:**

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