Cardiology Stethoscopes

Littmann®
Cardiology IV™
Stethoscope

3M

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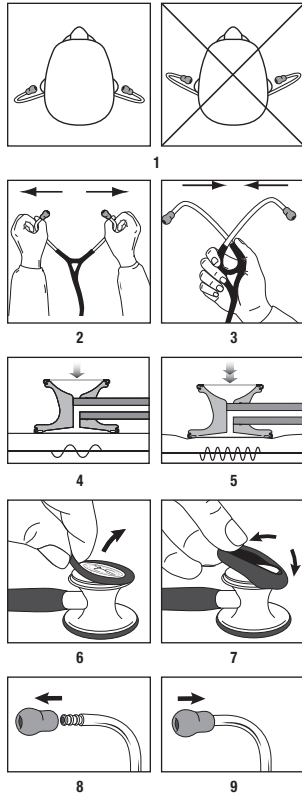
3M™ Littmann® Cardiology IV™ Stethoscope User Manual

en

Intended Use: The 3M™ Littmann® Cardiology IV™ Stethoscope is intended for use by healthcare professionals for medical diagnostic purposes only. It can be used for auscultation of heart, lung, and other body sounds.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Instructions for Use (see Figures pp. 1):

- **Place the Stethoscope in Your Ears:** The earbuds should point forward toward your nose (Figure 1).
- **Adjust the Headset Tension for Comfort and Sound Quality:** Pull the earbuds apart (Figure 2) or squeeze them together (Figure 3).
- **Select the Active Side of the Chestpiece:** A mark on the stem identifies which side of the chestpiece is acoustically active. Rotate the chestpiece to change which side is active.
 - Use the large side of the chestpiece for adults. Use the small side for pediatric or thin patients, around bandages, and for carotid assessment.
- **Listen to Low and High Frequency Sounds with Littmann Tunable Diaphragms:**
 - To emphasize lower frequency sounds, use light pressure against the patient (Figure 4).
 - To emphasize higher frequency sounds, use firm pressure against the patient (Figure 5).

• Convert to a Traditional Open Bell:

To remove the diaphragm from the small side, hold the large side of the chestpiece in one hand. While wearing an exam glove on your other hand, use your gloved thumb to peel the pediatric diaphragm off the small side of the chestpiece (Figure 6). Install the non-chill sleeve on the open bell (Figure 7).

Cleaning, Disinfection and Storage

Clean between exams using a 70% isopropyl alcohol wipe or disposable wipe with soap and water. A 2% bleach solution may be used to disinfect your stethoscope; however, the tubing may become discolored after exposure to bleach.

The diaphragms, non-chill bell sleeve, and earbuds may be removed for cleaning (Figure 6, 7 and 8). Ensure all parts and surfaces are dry before reassembly. To attach the earbuds, push the *small* side firmly onto the eartube until it snaps fully into place (Figure 9). To prevent ear canal injury, be sure earbuds are firmly attached to the headset before inserting them into your ears.

To prevent stethoscope damage, do not: immerse your stethoscope in any liquid, subject your stethoscope to any sterilization process, or store in extreme heat. To prevent staining of stethoscope tubing, avoid contact with pens, markers, newsprint, or other printed material. It is good practice to wear your stethoscope over a collar whenever possible.

Disposal: Dispose of contents/container in accordance with the local/regional/national/international regulations.

Littmann Stethoscope Service and Warranty Program: The Littmann Cardiology IV stethoscope is warranted against any defects in material and workmanship for a period of seven (7) years. Within the warranty period, repairs will be made without charge upon the return of the stethoscope to 3M, except in cases of obvious abuse or accidental damage. For service and repair in the USA, please visit www.littmann.com/service or call 1-800-292-6298. If you are outside of the USA, please visit www.littmann.com for the contact information of your local 3M office.

For more information, visit www.littmann.com.

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Made in USA with globally sourced materials.

 3M Company
2510 Conway Ave., St. Paul, MN 55144 USA
www.littmann.com
1-800-228-3957 (USA only)

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 **3M Deutschland GmbH**
Health Care Business
Carl-Schurz-Str. 1
41453 Neuss, Germany

Requester: Susan Barker Creator: 26183 File Name: 34872470009.indd Structure #: SS-40090 Date: 11/13/19	Printed Colors – Front: 
	Printed Colors – Back: 
	Match Colors:
<p>This artwork has been created as requested by 3M. 3M is responsible for the artwork AS APPROVED and assumes full responsibility for its correctness.</p>	

Scale:  1 Inch

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3M, Littmann, the L Littmann logo, the L, and Cardiology IV are trademarks of 3M. Used under license in Canada.
© 2019, 3M. All rights reserved./Tous droits réservés.